

## DESCOLIND CARE WIPES

Skin cleansing wipes



### PRODUCT DESCRIPTION

- With panthenol and allantoin for gentle patient washing
- Practical and quick to apply
- Microwaveable
- Also suitable for stoma care and cleaning the stoma area

### RANGE OF APPLICATION

Professional cleansing of the skin, especially for immobile patients.

#### Application


Remove wipes from the packaging and rub the area of the skin to be cleansed. Use a new wipe for each new area of the skin according to the overall body washing routine.

#### Application notes

To be warmed up in the microwave before use if necessary (600 W - 20 s). Microwave times may differ. Before using the wipes, ensure that they are not too hot. Do not reheat packs that have already been opened. Wipes are intended for single use.

#### Ingredients

AQUA, PROPYLENE GLYCOL, PANTHENOL, ALLANTOIN, COCO-GLUCOSIDE, CO-CAMIDOPROPYL BETAINE, ETHYLHEXYLGLYCERIN, CITRIC ACID, SODIUM CITRATE, SODIUM CHLORIDE, SODIUM BENZOATE, PARFUM.

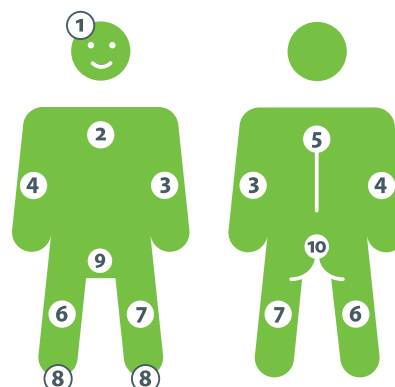

Product	Single unit	Content	Size	Delivery unit	REF
DESCOLIND CARE WIPES	pack	10 wipes	20 x 30 cm	24	00-604DW1-T010-01

pH 0 1 2 3 4 5 6 7 8 9 10 11 12 13 14