

Instruction for reprocessing of touchless dispenser system:

RX 5 T



A complete setup of the dispenser system requires several steps to be followed. Those steps, described in detail in the subsequent document, are structured as follows:

- Preparative measures
- Cleaning and disinfection
- Frequency of reprocessing and additional measures

Preparative measures

For protection gloves should always be worn – in accordance with the indication “contact with potentially infectious materials”.



Remove drip tray, bottle and pump.



Open battery compartment and remove batteries.



Cleaning and Disinfection

Wipe the dispenser and the drip tray with a damp cloth and dry with a disposable paper towel.



Wipe the dispenser and the drip tray with a suitable surface disinfectant.



Clean the pump, including pump outlet using warm water.



Cleaning and Disinfection

Dry the pump and pump outlet with a disposable paper towel.



Pump several times with a suitable disinfectant.



Clean the sensor with a small brush.



Frequency of reprocessing and additional measures

Insert batteries, pump, bottle and drip tray.



Prime the dispenser pump.



OPHARDT recommendation: Reprocessing twice a year minimum.
Please note: The dispenser is not autoclavable and can not be cleaned in the dishwasher (no mechanical reprocessing).